



Dog Tracheal Tube

- A special tracheal tube designed for dog.
- This dog tracheal tube is used to deliver anesthestic gas and/or oxygen to patients.
- Designed to sit in the pharynx, and can avoid many problems caused by tubes such as laryngeal and tracheal trauma.



Dog Tracheal Tube



Step1: Choose the correct tube.



Step2: Use a laryngoscope to examine the mouth and pharynx, then remove any visible foreign objects or excessive secretions.

Feature and Benefits

- Suitable for small dogs and adult large dogs
- Easy to place-particularly useful in emergencies
- No airway narrowing, no increase in airway resistance and protect airway from reflux
- Low dead-space connector
- Excellent seal presure allowing assisted ventilation but no environmental pollution of anesthetic gases
- No tracheal trauma and no laryngeal damage/spasms
- Safer, smoother recoveries and no post-operative gagging or coughing



Step3: Grasp the tongue with a dry gauze to open the mouth and then insert the tube gently into the pharynx (sterile lube can be used on the edges of tube to slide easily).



Step4: Secure the tie gauze around the tube and tie to pet. Lay the pet down and connect the tube to the anesthesia machine.

	Head shape - Brachyce phalic	Lean weight of dog	Head shape - Mesocephalic	Lean weight of dog	Head shape - Dolichocephalic	Lean weight of dog	Size/ Product code
		6-10kg 13-22lb		4-7kg 9-15lb		2-4kg 4.5-9lb	D3=D20003
		9-15kg 20-33lb		6-10kg 13-22lb		4-7kg 9-15lb	D4=D20004
		14-20kg 30-45lb		9-15kg 20-33lb		6-10kg 13-22lb	D5=D20005
		19-26kg 42-57lb		14-20kg 30-45lb		9-15kg 20-33lb	D6=D20006

Remark: this is only for your reference. The appropriate size should be determined by the professional veterinarians based on the practical situation.